
# ***Automatic Digital Blood Pressure Arm Monitor***


Model 01-5021 (adult)

Model 01-5022 (large adult)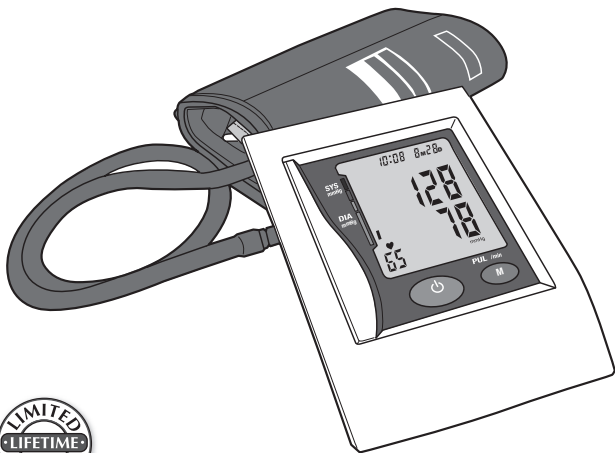


**INSTRUCTION MANUAL • ENGLISH & ESPAÑOL**


**STOP!**  
**PLEASE ENSURE YOU HAVE ALL OF THE FOLLOWING**  
**COMPONENTS BEFORE USING YOUR DIGITAL**  
**BLOOD PRESSURE MONITOR**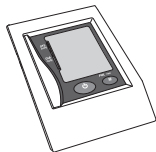

Digital Monitor

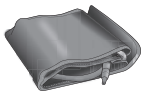
Adult Arm Cuff  
or Large Adult Arm  
Cuff

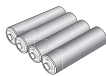4 AA Batteries

Canvas Storage Bag

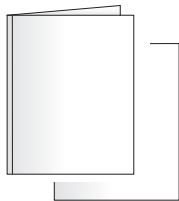Instruction Manual  
Quick Start Guide  
Blood Pressure Reading Tracking Log  
FAQ Information Sheet  
Replacement Parts Order Form

**IF YOU ARE MISSING ANY PARTS, INCLUDING INSERTS OR  
INSTRUCTION MANUALS, DO NOT RETURN TO PLACE OF  
PURCHASE. CONTACT CUSTOMER CARE AT 866-326-1313.**

---

# INDEX

---

Introduction .....	4
Digital Blood Pressure Monitor Features .....	5
Getting Started.....	6
Battery Installation / AC Power .....	7
Setting Date and Time.....	8
Applying Your Blood Pressure Cuff.....	9-10
Taking Your Blood Pressure Reading .....	11-12
Interpreting your Results.....	13
Memory Functions .....	14-15
Care and Maintenance .....	15
Display Symbols.....	16
Troubleshooting .....	16
FCC Statement.....	17
Specifications .....	18
Warranty .....	19
Instrucciones en Español.....	21-40

---

**Toll-Free Customer Care Help Line: 1-866-326-1313**  
**Monday – Friday 8:30 a.m. – 4:30 p.m. CST**

Distributed by  
Veridian Healthcare  
1465 S. Lakeside Drive  
Waukegan, IL 60085  
[www.veridianhealthcare.com](http://www.veridianhealthcare.com)

Made in China

#93-1045 04/10  
©2010 Veridian Healthcare, LLC

---

# INTRODUCTION

---

Veridian Healthcare recommends that you first seek the advice and recommendation of your physician or healthcare professional when using home diagnostic devices, including blood pressure monitors.


Automatic digital blood pressure monitors use the oscillometric method to electronically measure your blood pressure. The monitor detects your blood's movement through the artery in your arm and converts the movements into a digital reading. The oscillometric method does not require a stethoscope, making the monitor ideal for home use.

Blood pressure readings determined with this device are equivalent to measurements obtained by a trained healthcare professional using the cuff/stethoscope auscultation method, within the limits prescribed by the American National Standard for Electronic or Automated Sphygmomanometers.

Thank you for purchasing a Veridian Healthcare Automatic Digital Blood Pressure Monitor. With proper care and use, your monitor will provide you with many years of reliable readings.

**PLEASE READ THIS INSTRUCTION MANUAL  
COMPLETELY BEFORE OPERATING THIS UNIT.**

# BLOOD PRESSURE MONITOR FEATURES

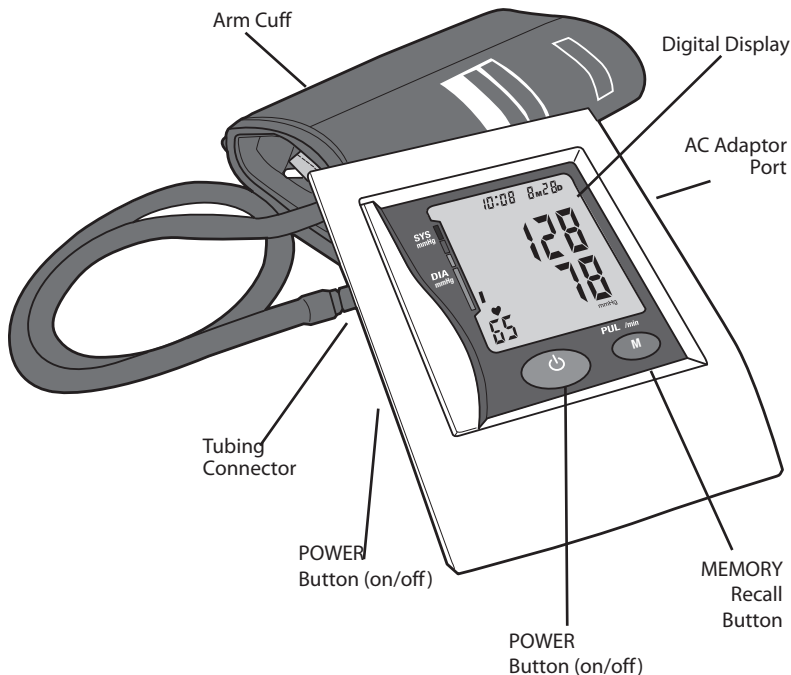**DETAILS OF THE DISPLAY AND INTERPRETATION OF THE SYMBOLS ARE INCLUDED THROUGHOUT THIS MANUAL.**

---

# GETTING STARTED

---

## Before Taking a Measurement

1. Your readings should only be interpreted by your physician or healthcare professional with access to your individual medical history. Regular use of a home blood pressure monitor will allow you to track and record your readings for discussion with your physician.
2. Conduct your measurement in a quiet place while seated in a relaxed position. Rest for 15 minutes before taking your reading. If an error occurs or you desire to take a second reading, allow 15 minutes between readings for your blood vessels to return to normal.
3. Avoid smoking, eating, taking medication, alcohol, physical activity or any other stressful activity for 30 minutes prior to taking a reading.
4. Always remove any jewelry or constrictive clothing that may interfere with the cuff placement.
5. Keep yourself and the monitor still during measuring; do not talk during the reading.
6. It is recommended that you take your readings at the same time each day to better monitor any indications in your results.
7. Record your daily measurement on the included chart or some other written document to share with your physician.

# BATTERY INSTALLATION / AC POWER

## This unit comes complete with 4 AA batteries.

It is necessary to replace the batteries when the Low Battery symbol appears on the display or when the display does not turn on after the POWER button is pressed. 

### Replacing Battery, Fig. 1

1. Press down and lift the cover.
2. Insert or replace 4 AA batteries into the battery compartment, ensuring to match the indicated polarity symbols. Always use new batteries.
3. Replace the battery cover.
4. It is recommended to remove the batteries if the unit will not be used for an extended period of time.

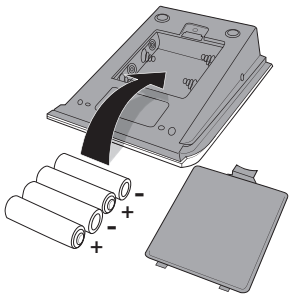
Fig. 1

### AC Power Adapter

An AC adapter is available separately for battery-free operation. The AC adapter plugs into the right side of the unit, Fig. 2.

Please remember to practice common sense electrical safety precautions while using the adapter.

To order an AC adapter, please see the enclosed order form.

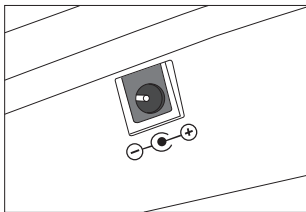
Fig. 2

AC Adapter Output: 6V DC 600mA

# SETTING DATE/TIME

**IT IS NECESSARY TO SET THE DATE AND TIME FOR THE MONITOR EVERY TIME BATTERIES ARE INITIALLY INSTALLED OR REPLACED.**

**DATE AND TIME WILL DEFAULT TO FACTORY SETTINGS, 12:00 A.M. 1M 1D, IF YOU DO NOT CHOOSE TO SET THE CORRECT INFORMATION.**

1. When the unit is off, press and release both the  and M button at the same time. The month will begin to flash, Fig. 1.
2. Press the M button to change until the desired number appears. To scroll through the numbers quicker, hold down the M button as the numbers advance.
3. Press the  button to confirm the month entry.
4. Next, the date will blink. Repeat steps 2 and 3 to set the date, then hours, then minutes. This unit does not track the year.
5. After setting the minutes, the unit will automatically exit out of the date/time setting mode.

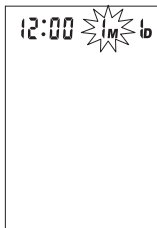
Fig. 1

When the monitor is off, the date and time will still appear at the top of the screen.

Time is displayed in 24hr Military Time. This chart explains the time conversion.

## MILITARY TIME = REGULAR TIME

1:00 = 1am	7:00 = 7am	13:00 = 1pm	19:00 = 7pm
2:00 = 2am	8:00 = 8am	14:00 = 2pm	20:00 = 8pm
3:00 = 3am	9:00 = 9am	15:00 = 3pm	21:00 = 9pm
4:00 = 4am	10:00 = 10am	16:00 = 4pm	22:00 = 10pm
5:00 = 5am	11:00 = 11am	17:00 = 5pm	23:00 = 11pm
6:00 = 6am	12:00 = 12pm	18:00 = 6pm	24:00 = 12am

# APPLYING YOUR BLOOD PRESSURE CUFF

**PROPER CUFF FIT AND PLACEMENT IS CRITICAL IN OBTAINING ACCURATE BLOOD PRESSURE MEASUREMENTS. PLEASE READ THESE INSTRUCTIONS CAREFULLY AND CONSIDER CONSULTING WITH YOUR PHYSICIAN FOR FURTHER DIRECTION.**

Before applying your blood pressure cuff, be sure you have selected the appropriate size cuff.

To determine the proper cuff size, measure the circumference of your left upper arm at the point midway between inside bend of your elbow and your shoulder, Fig 1.

**Adult Cuff** fits arm circumference 11-3/4" - 16-1/2" (30-42 cm)

**Large Adult Cuff** fits arm circumference 16-1/2" - 18-7/8" (42-48 cm)

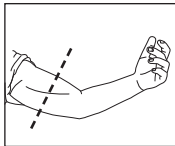
Fig. 1

Additional cuff sizes are available for order on the enclosed order form.

If for any reason you are unable to or should not use your left arm, please modify the instructions for cuff application to your right arm. Your physician can help you identify which arm is best for you to take measurements from.

1. The cuff should not be plugged into the monitor until after the cuff is applied to your arm.
2. Remove any constrictive clothing or jewelry that may interfere with cuff placement.
3. Be seated at a table or desk with your feet flat on the floor.
4. Position the cuff on a solid surface with the tubing facing up and away from you. The metal ring/bar on the cuff should be to the left of the tubing.

## APPLYING YOUR BLOOD PRESSURE CUFF

5. Open the cuff by pulling or rolling the bottom of the cuff to the right, Fig 2. This should open the cuff without fully unrolling it, creating a cylinder. Do not extend the cuff beyond the metal bar.
6. Insert your left arm into the created cuff cylinder. Position the index mark  $\Phi$  over the main artery on the inside of your arm, Fig 3, 4.
7. The bottom edge of the cuff should be positioned approximately one inch above the elbow joint, Fig 5.
8. The cuff should fit comfortably, yet snugly around your arm. You should be able to insert one finger easily between your arm and the cuff.

The cuff size is suitable for use when the vertical index mark  $\Phi$  is within the horizontal OK range. A different size cuff is needed if the 'index' mark is outside the OK range.

If you are not comfortable with applying your cuff, please seek the assistance of another member of your household or work with your physician to practice the cuff application. Incorrectly applied cuffs may result in inaccurate readings.

Fig. 2

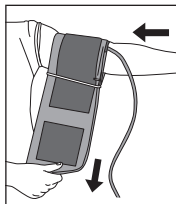
Fig. 3

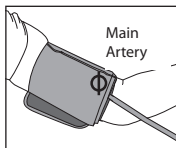
Fig. 4

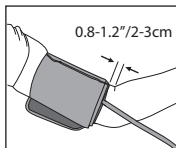
Fig. 5

# TAKING YOUR BLOOD PRESSURE READING

Please read the preceding portions of this manual prior to taking your first reading.

Reminder, it is important to avoid smoking, eating, taking medication, alcohol consumption or physical activity 30 minutes prior to taking a reading.

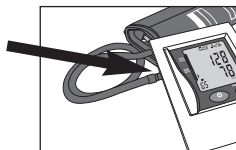
Fig. 1

1. Position the monitor on a flat, stable surface with the digital display in view.
2. Insert the cuff tubing connector into the port on the left side of your monitor, Fig 1.
3. Rest your elbow on a solid surface with your palm facing upward. Elevate your arm so that the cuff is at the same level as your heart, Fig 2. Relax your left hand.
4. Press the  (POWER) button to turn the unit on; the unit will run a self-test, Fig 3.
5. The unit will briefly display the last reading value for that memory bank. If there is no previous measurement, the unit will display '0' for all of the reading values, Fig 4.
6. The cuff will automatically inflate to approximately 190mmHg or 40 mmHg higher than the last systolic pressure reading store in memory.

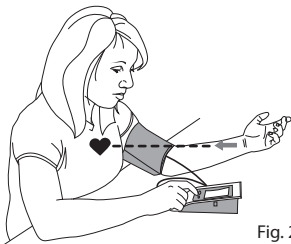
Fig. 2

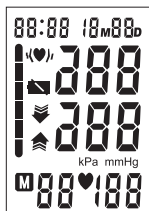
Fig. 3

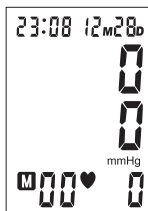
Fig. 4

# TAKING YOUR BLOOD PRESSURE READING

**YOU CAN STOP THE INFLATION OR DEFLATION PROCESS AT ANY TIME BY PRESSING THE POWER BUTTON.**

- When the inflation has reached optimum level, the display will begin to show the decreasing pressure while you feel the pressure of the cuff decrease.
- Your blood pressure measurement and pulse will display simultaneously on the screen, Fig 5. Both the systolic and diastolic readings will display at the same time.
- The Hypertension Indicator will indicate your reading range in the left side of the display, Fig 6. See page 14 for a chart interpreting the Hypertension Indicator results.
- If an irregular heartbeat was detected during the reading, the Irregular Heartbeat Detector will appear on the display, Fig 7. See page 14 for more information on the Irregular Heartbeat Detector feature.
- The reading will automatically be stored in memory, up to 60 readings.
- Select **⏻**(POWER) to turn the unit off and conserve energy and battery life. The unit will automatically shut-off and revert to time/date mode after approximately 1 minute.
- Disconnect the cuff and tubing from the monitor prior to storing.

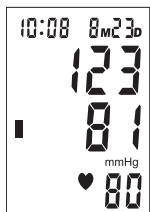
Fig. 5

Fig. 6

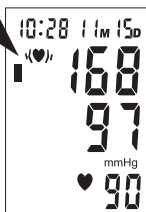
Fig. 7

# INTERPRETING YOUR RESULTS

## Irregular Heartbeat Detection


Your digital blood pressure monitor features an Irregular Heartbeat Detector. This feature allows users to accurately monitor blood pressure even if an irregular heartbeat should occur. When an irregular heartbeat is detected, the IHB icon will appear on the display.

Please consult with your physician or trained healthcare professional for further information regarding an irregular heartbeat and if this symbol appears frequently.

## Hypertension Indicator

This unit features our unique Hypertension Indicator. The World Health Organization has established globally accepted standards for the assessment of high or low blood pressure readings. The below chart should be considered only as a guideline, always consult with your physician or health care professional to interpret your individual results.

The Hypertension Indicator, located on the left side of the display, will identify your reading within a range in accordance with the guidelines set by the World Health Organization (WHO).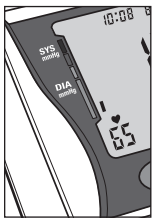

The Hypertension Indicator is located on the left side of the display

**Stage 3 Hypertension**  
(red bar)  
Systolic pressure:  $\geq 180$  mmHg  
or  
Diastolic pressure:  $\geq 110$  mmHg

**Stage 2 Hypertension**  
(orange bar)  
Systolic pressure:  $\geq 160$ -179 mmHg  
or  
Diastolic pressure:  $\geq 100$ -109 mmHg

**Stage 1 Hypertension**  
(yellow bar)  
Systolic pressure:  $\geq 140$ -159 mmHg  
or  
Diastolic pressure:  $\geq 90$ -99 mmHg

**High Normal**  
(green bar)  
Systolic pressure:  $\geq 130$ -139 mmHg  
or  
Diastolic pressure:  $\geq 85$ -89 mmHg

**Normal**  
(green bar)  
Systolic pressure:  $\geq 120$ -129 mmHg  
or  
Diastolic pressure:  $\geq 80$ -84 mmHg

**Optimal**  
(green bar)  
Systolic pressure:  $< 120$  mmHg  
or  
Diastolic pressure:  $< 80$  mmHg

# MEMORY FUNCTION

## Recalling Measurements in Memory

You can recall up to 60 total measurements in memory. These measurements can be shared with your physician or trained healthcare professional.

1. Press and release the M (MEMORY) button. The unit will show how many measurements are stored within the bank, Fig 1.
2. Press the M (MEMORY) button again to scroll through the saved measurements. Measurements will appear on the display from oldest to most current. All results for a given measurement will display, including measurement results, pulse rate, Hypertension Indicator icon and Irregular Heartbeat alert (if applicable), Fig 2. After you've scrolled through all the readings, the unit will automatically turn off.
3. When the number of readings exceeds 60, the oldest data will be replaced with the new record.

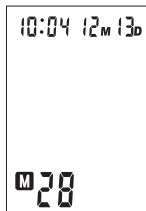
Fig. 1

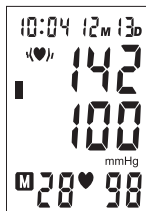
Fig. 2

---

## MEMORY FUNCTION

---

### Clearing Measurements from Memory

1. Press and release the M (MEMORY) button. The unit will display how many measurements are stored within the bank.
2. Press and hold down the M (MEMORY) button until two horizontal lines flash, Fig. 3. This indicates that all measurements have been erased.

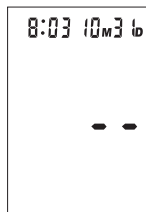
Fig. 3

---

## CARE & MAINTENANCE OF MONITOR

---

- Disconnect the cuff and tubing from the monitor prior to storing.
- Do not use any liquids on the monitor or cuff.
- Use a soft, dry cloth to clean your monitor.
- Do not store in direct sunlight, dust or excessive humidity.
- Avoid extreme temperatures.
- Do not disassemble the monitor or cuff.

# DISPLAY SYMBOLS

Symbol	Meaning	Symbol	Meaning
	Low battery		An irregular heartbeat was detected during measurement
	Pulse rate per minute		The cuff is not properly wrapped or has too much air left in it. The arrow will cease flashing when the unit is ready for measurement
	An error has occurred during measurement		A measurement error has occurred and the measurement should be attempted again.
	Reading on display is a reading from a memory bank		

# TROUBLESHOOTING

Problem	Possible Cause	Solution
Display shows an abnormal result	Cuff position was not correct or was not tight enough	Apply the cuff properly and attempt a new measurement
	Body posture was not correct during measurement	Review the 'Taking a Measurement' portion of this manual and attempt a new measurement
	Speaking, excessive movement, anger, excited or nervous anxiety during measurement	Wait a period of time and attempt a new measurement after calm has returned; do not speak or move during measurement
Display is blank when power is on	Batteries may be expired or installed improperly	Check the polarity of the batteries and reinstall if necessary; replace batteries with new batteries

---

# FCC STATEMENT

---

## **NOTE:**

### POTENTIAL FOR RADIO/TELEVISION INTERFERENCE (for U.S.A. only)

This product has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC rules.

These limits are designed to provide reasonable protection against harmful interference in a residential installation. The product generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If the product does cause harmful interference to radio or television reception, which can be determined by turning the product on and off, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna
- Increase the separation between the product and the receiver.
- Connect the product into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

### POTENTIAL FOR RADIO/TELEVISION INTERFERENCE (for Canada only)

This digital apparatus does not exceed the Class B limits for radio noise emissions from digital apparatus as set out in the interference-causing equipment standard entitled "Digital Apparatus", ICES-003 of the Canadian Department of Communications.

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Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

# PRODUCT SPECIFICATIONS

---

<b>NAME</b>	Automatic Premium Digital Blood Pressure Arm Monitor
<b>MODEL NUMBER</b>	01-5021 (adult); 01-5022 (large adult)
<b>DISPLAY SYSTEM</b>	Digital display/LCD
<b>MEASUREMENT METHOD</b>	Oscillometric
<b>POWER SOURCE</b>	4 "AA" batteries
<b>MEASURING RANGE</b>	Pressure: 0-300 mmHg; Pulse: 30-180 beats/minute
<b>ACCURACY</b>	Pressure: $\pm 3$ mmHg; Pulse: $\pm 5\%$ of reading
<b>INFLATION</b>	Pump Drive
<b>DEFLATION</b>	Automatic Pressure Release Valve
<b>MEMORY</b>	Built-in memory enabling display of up to 60 measurements
<b>AUTOMATIC SHUT-OFF</b>	Approximately 1 minute after last button operation
<b>OPERATION ENVIRONMENT</b>	Temperature 41°F – 104°F (5°C – 40°C); Humidity < 80%
<b>STORAGE ENVIRONMENT</b>	Temperature -4°F – 131°F (-20°C – 55°C); Humidity < 95%
<b>MONITOR DIMENSIONS</b>	4-1/8" x 6" x 1-3/4"
<b>WEIGHT</b>	12.7 oz. (without batteries)
<b>CUFF ARM CIRCUMFERENCE</b>	Adult size cuff fits arm circumference: 11-3/4" - 16-1/2" (30-42 cm) Large adult size cuff fits arm circumference: 16-1/4" - 18-7/8" (42-48 cm)
<b>ACCESSORIES</b>	Cuff, detailed guidebook, quick start guide, blood pressure log, four "AA" batteries, storage case
<b>OPTIONS (AVAILABLE SEPARATELY)</b>	Small adult size cuff fits arm circumference: 8-5/8" - 11-3/4" (22-30 cm), AC Adapter

***Specifications are subject to change without notice***

---

# LIFETIME LIMITED WARRANTY

---

Congratulations on your purchase of a Veridian Digital Blood Pressure Monitor. Your Digital Blood Pressure Monitor is covered by the following limited warranty commencing upon the date of purchase, and subject to the following terms and conditions:

Veridian Healthcare warrants that its Digital Blood Pressure Monitor will be free from defects in materials and workmanship under normal consumer usage for as long as the original purchaser owns the product.

Periodic maintenance, repair and replacement of parts due to normal wear and tear are excluded from coverage. Defects or damage that result from: (a) improper operation, storage, misuse or abuse, accident or neglect, such as physical damage (cracks, scratches, etc.) to the surface of the product resulting from misuse; (b) contact with liquid, water, rain, extreme humidity or heavy perspiration, sand, dirt or the like, extreme heat, or food; (c) use of the Digital Blood Pressure Monitor for commercial purposes or subjecting the Digital Blood Pressure Monitor to abnormal usage or conditions; or (d) other acts which are not the fault of Veridian, are excluded from coverage. This warranty does not cover batteries or other power sources that may be provided with, or used with the Digital Blood Pressure Monitor.

If the Digital Blood Pressure Monitor fails to conform to this limited warranty, return the Digital Blood Pressure Monitor postage prepaid along with \$5.00 for return shipping, handling and insurance to: Veridian Healthcare, Attn: Repair Department, 1465 S. Lakeside Drive, Waukegan, IL, 60085. When returning a product, please also include: (i) a copy of your receipt, bill of sale or other comparable proof of purchase; (ii) a written description of the problem; and (iii) your name, address and telephone number. Carefully package the product to avoid any damage that may occur while in transit; shipping insurance with returned receipt is recommended. At our option, Veridian Healthcare will repair or replace the unit found to be defective in materials or workmanship under normal consumer usage. The purchaser will be notified of any additional repairs required prior to completing the repair, and will be responsible for parts charges, if any, and repair charges not covered by this limited warranty.

EXCEPT AS PROVIDED FOR IN THIS LIMITED WARRANTY, ALL EXPRESS AND IMPLIED WARRANTIES AND CONDITIONS ARE DISCLAIMED, INCLUDING WITHOUT LIMITATION THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. THE REPAIR OR REPLACEMENT AS PROVIDED UNDER THIS LIMITED WARRANTY IS THE EXCLUSIVE REMEDY OF THE CONSUMER, AND IS PROVIDED IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. IN NO EVENT SHALL VERIDIAN HEALTHCARE BE LIABLE, WHETHER IN CONTRACT OR TORT (INCLUDING NEGLIGENCE) FOR DAMAGES IN EXCESS OF THE PURCHASE PRICE OF THE PRODUCT, OR FOR ANY INDIRECT, INCIDENTAL, SPECIAL OR CONSEQUENTIAL DAMAGES OF ANY KIND, OR FOR DAMAGES TO, OR LOSS OF, OTHER PROPERTY OR EQUIPMENT OR PERSONAL INJURIES TO THE FULL EXTENT THESE DAMAGES MAY BE DISCLAIMED BY LAW.

**Some states and jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitation on the length of an implied warranty, so the above limitations or exclusions may not apply to you. This warranty gives you specific legal rights, and you may also have other rights that vary from state to state or from one jurisdiction to another.**

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